# 2

## **BOLTEX INERTIAL, TONIFICA Y FORTALECE TU SUELO PÉLVICO**

**BOLTEX INERTIAL,  
TONES AND STRENGTHENS  
YOUR PELVIC FLOOR**

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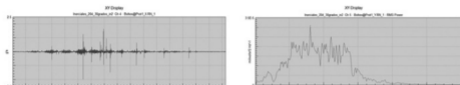
Boltex Inertial is a medical device designed to tone and recondition women's pelvic floor muscles. It has an **exclusive and patented design** based on a set of hexagonal cells on the internal surface of the sphere, which give it extraordinary properties that help to provide **effective stimulation of the pelvic floor muscles**, regardless of its position after having been inserted into the vagina and of the user's anatomical and morphological characteristics.

The efficiency of an inertial device lies in its capacity to tone the PFM. The accelerometry graph shows the comparison between Boltex Inertial, in which the interior sphere knocks against the shell (Top graph) and an inertial device in which the interior sphere rolls instead of knocks (bottom graph, other models on the market). After knocking, a vibration is produced, which leads to a reflex toning action in the perineal muscles.

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
Gráfica superior (Boltex Inertial) / Top graph (Boltex Inertial) / Graphique supérieur (Boltex Inertial)

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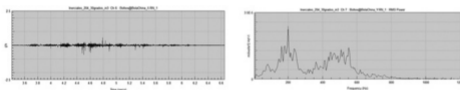
Likewise, the mechanical stimulation with Boltex, combined with voluntary contraction exercises (Kegel) make the toning more effective.

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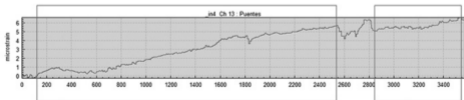The first part of the graph shows the progressive toning of the pelvic floor muscles with mechanical vibratory stimulation; in the second part, the mechanical stimulus is stopped; and in the third part, mechanical stimulation is used again, along with voluntary contraction exercises (Kegel).

Boltex Inertial also helps you to know the tone level of your pelvic muscles. Should the device drop during use and need to be moved into the vaginal passage again, the degree of hypotonia is very mild. If it is pushed out shortly after being placed, the loss of tone is severe. The more you use it, the better tone you will achieve, and so it will be easier to hold it in.

During menstruation and ovulation, the device can have a tendency to drop more easily, due to hormonal changes. Be constant and persistent with the treatment.

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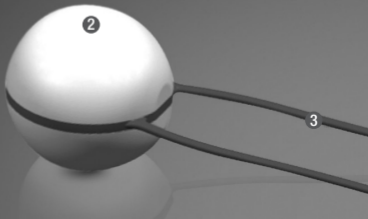
Boltex Inertial is a medical device that meets existing health regulations. The materials used have been tried and tested, and meet all biocompatibility requirements. Thanks to its features, it is ideal for assisting insertion and removal, ensuring it is easy and comfortable for the user and, in particular, guaranteeing contact between the surface of the sphere and the vaginal walls. It is totally hermetic with a smooth outer wall, preventing any build-up of dirt and bacteria and making it easier to clean. It is also comfortable and safe to use.

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### 1. INTERIOR BALL

Coated with silicone to prevent noise.

### 2. THE SPHEROID

It is made from PC/ABS, as this material is a good transmitter of vibration.

### 3. THE STRING

Makes it easier to extract from the vagina. It is made from TPE, which guarantees its medical quality and health and hygiene conditions.

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